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General information

- > 97% cotton and 3% elastane
- > Free of latex and medicines
- > Permeable to air and water, as well as breathable
- > Excellent adhesion in the water
- > Do not restrict the movement

Not to be used at

- > Fresh skin injuries or open wounds
- > Coagulation disorders
- > Vascular pathologies (thrombosis)
- > Acute, chronic skin diseases
- > Allergies to one of the ingredients in the tape or acrylic adhesive (first on a small Skin site test)
- > Malignant tumors
- > Pregnant women may only use the tape after use the language with the attending physician.
- > Not for patients in cancer treatment

If skin irritation occurs when applying the tape, gently remove the tape.

Precautions

At the body parts, which are difficult for the patient to reach, the tape of the appropriate specialist personnel are glued.

General notes on the installation of tape

- 1) Remove the skin before applying creamy residue and other contaminants.
- 2) Apply the tape only on dry and clean skin
- 3) Locate the appropriate muscle area
- 4) Measure the appropriate length of the tape
- 5) Round off the corners to prevent premature peeling
- 6) Detach the backing paper about 3-4cm from the tape
- 7) Glue the tape to the skin according to your condition
 - > The tape must be rubbed warm after the installation in order to remove the Activate adhesive
 - > The tape should be at least 30-60min before any sporting activities be created
 - > Dab the tape lightly to dry
- 8) The tape always in the direction of hair growth and not jerky remove.

storage

Store cool, dry and protected from direct sunlight between 8 ° C and 30° C

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Param GmbH