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### General information

- > 97% cotton and 3% elastane
- > Free of latex and medicines
- > Permeable to air and water, as well as breathable
- > Excellent adhesion in the water
- > Do not restrict the movement

### Not to be used at

- > Fresh skin injuries or open wounds
- > Coagulation disorders
- > Vascular pathologies (thrombosis)
- > Acute, chronic skin diseases
- > Allergies to one of the ingredients in the tape or acrylic adhesive (first on a small Skin site test)
- > Malignant tumors
- > Pregnant women may only use the tape after use the language with the attending physician.
- > Not for patients in cancer treatment

**If skin irritation occurs when applying the tape, gently remove the tape.**

### Precautions

At the body parts, which are difficult for the patient to reach, the tape of the appropriate specialist personnel are glued.

### General notes on the installation of tape

- 1) Remove the skin before applying creamy residue and other contaminants.
  - 2) Apply the tape only on dry and clean skin
  - 3) Locate the appropriate muscle area
  - 4) Measure the appropriate length of the tape
  - 5) Round off the corners to prevent premature peeling
  - 6) Detach the backing paper about 3-4cm from the tape
  - 7) Glue the tape to the skin according to your condition
- > The tape must be rubbed warm after the installation in order to remove the Activate adhesive
  - > The tape should be at least 30-60min before any sporting activities be created
  - > Dab the tape lightly to dry
  - 8) The tape always in the direction of hair growth and not jerky remove.

### storage

Store cool, dry and protected from direct sunlight between 8 ° C and 25 ° C  
Keep out of reach of children.

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3 Jahre

### Product description

PARAM Kinesiologie Sport Tape, for an alternative treatment free from drugs and other substances.

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### Param GmbH